

## SDS 5 Earwax Guard


"Removal"  
end

Used earwax guards  
removed at this end


Replacement earwax  
guards stored at this end

"Insertion"  
end

- C. To dispose of the used earwax guard, insert the **removal end** of the tool into the tip of the receiver. Carefully pull straight out to **remove the earwax guard**. Do not twist when removing

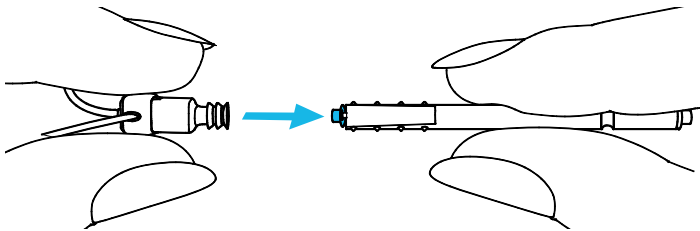

- D. Ensure the receiver is empty and a new earwax guard is visible at the **insertion end** of the tool. Gently push the **new earwax guard** into the tip of the empty receiver until the outer ring is aligned

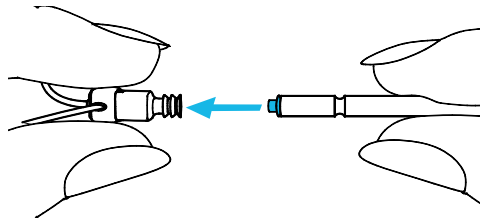

- E. **Reattach** the silicone dome onto the receiver and dispose of the old earwax guard and stick